PRONTO[®]-Short Extraction Catheter Model 5030 Instructions For Use

USA CAUTION

Federal (USA) law restricts this device to sale by or on the order of a physician.

CAUTION

The PRONTO-Short extraction catheter procedure should be performed by physicians with adequate training in the use of the device. The PRONTO-Short is provided sterile for single use only. Reuse of single-use device creates a potential risk of patient or user infections. Contamination of the device may lead to illness or serious patient injury.

DEVICE DESCRIPTION

The PRONTO-Short extraction catheter is a dual lumen, over-the-wire (OTW) catheter with related accessories. The catheter is designed to be delivered through a 6F or larger introducer sheath over a 0.018" (0.45mm) guidewire. The larger lumen allows for the removal of thrombus by use of the included syringes through the extension line and 3-way stopcock. The catheter has a rounded distal tip with a protected, sloped opening of the extraction lumen to facilitate advancement of the catheter into the blood vessel or vascular graft and maximize extraction of thrombus through the extraction lumen.

The catheter has a radiopaque marker band located approximately 3mm from the distal tip. The proximal end of the catheter incorporates a hemostatic Y-junction that allows for the attachment of the catheter to the included extension line, 3-way stopcock and syringes; and can be tightened down on the guidewire to prevent blood leakage. A 70 micron filter basket is included for assistance in filtering the blood removed during the procedure for laboratory analysis of any thrombus.

INDICATIONS

The PRONTO-Short extraction catheter is indicated for:

- the removal/aspiration of embolic material (thrombus/debris) from vessels of the arterial system.
- the removal/aspiration of thrombus from clotted synthetic dialysis grafts and arteriovenous fistulas.

CONTRAINDICATIONS

The PRONTO-Short catheter is contraindicated in:

- Vessels < 2.5mm in diameter
- the removal of fibrous, adherent or calcified material (e.g. chronic clot, atherosclerotic plaque)

WARNINGS

Do not use the PRONTO-Short catheter for the delivery or infusion of diagnostic, embolic or therapeutic materials into blood vessels as it has not been designed for these uses.

Do not use the syringes, extension line, stopcock, and filter basket inside the human body.

If flow into the syringes stops or is restricted, DO NOT discontinue suction; DO NOT attempt to flush the extraction lumen while the catheter is still inside the patient's vasculature. Intravascular thrombus delivery, a thromboembolic event and/or serious injury or death may result. Remove the catheter while the suction is on, and once outside the patient, either flush the extraction lumen or use a new catheter.

Do not move the PRONTO-Short catheter across the arterial or venous anastomosis of a thrombus-

laden AV fistula or synthetic dialysis graft. This may result in thrombosis of the arterial or venous system, acute arterial occlusion, or pulmonary embolism.

COMPLICATIONS

As with all catheterization procedures, complications with the PRONTO-Short catheter may occur. These may include:

- local or systemic infection
- local hematomas
- intimal disruption
- arterial dissection
- perforation and vessel rupture
- arterial thrombosis
- distal embolization of blood clots and plaque
- pulmonary embolism
- arterial spasm
- arteriovenous fistula formation
- catheter fracture with tip separation and distal embolization

PRECAUTIONS

The PRONTO-Short catheter deployment procedure should be performed by physicians thoroughly trained in percutaneous, intravascular techniques and procedures.

Do not use the PRONTO-Short catheter if the packaging has been damaged.

Inspect the catheter prior to use for any bends or kinks. Do not use a damaged catheter because vessel damage and/or inability to advance or withdraw the catheter may occur.

Check that all fittings are secure so that air is not introduced into the extension line or the syringes during extraction.

Exercise care in handling of the catheter during a procedure to reduce the possibility of accidental breakage, bending or kinking.

When the catheter is in the body, it should be manipulated only under fluoroscopy. Do not attempt to move the catheter without observing the resultant tip response.

Never advance or withdraw an intravascular device against resistance until the cause of the resistance is determined by fluoroscopy. Movement of the catheter or guidewire against resistance may result in separation of the catheter or guidewire tip, damage to the catheter, or vessel perforation.

Precautions to prevent or reduce clotting should be taken when any catheter is used in the vascular system. Use of systemic heparinization and heparinized sterile solutions should be considered.

CLINICAL PROCEDURE

following instructions provide technical direction but do not obviate the necessity of formal training in the use of the PRONTO-Short catheter. The techniques and procedures described do not represent ALL medically acceptable protocols, nor are they intended as a substitute for the physician's experience and judgment in treating any specific patient.

Each PRONTO-Short extraction catheter includes the following components:

- Single-use, disposable catheter (40cm)
- Extension line (15cm) with attached 3-way stopcock
- 2 - 30ml syringes with locking plunger
- Filter basket (70 micron mesh) for filtering thrombus after extraction

English/Instructions for Use	1
Český/Návod k použití	2
Dansk/Bruksanvisning.....	4
Nederlands/Gebruiksaanwijzing.....	5
Suomi/käyttöohjeet.....	6
Français/Mode d'emploi	8
Deutsch/Gebrauchsanweisung	9
Ελληνικά/Οδηγίες χρήσης	11
Italiano/Istruzioni per l'uso.....	13
Polski/Instrukcja użytkowania.....	14
Português/Instruções de Utilização...	15
Español/Instrucciones de uso	17
Svensk/Bruksanvisning på svenska ..	18
Türkçe/Kullanma Talimatı.....	20

Vascular Solutions, Inc.
6464 Sycamore Court
Minneapolis, MN 55369 USA
(888) 240-6001 USA
(763) 656-4300
(763) 656-4250
www.vasc.com

Vascular Solutions Zerusa Limited
208 Business Innovation Centre
NUI Galway
Newcastle Road
Galway Ireland
+353 91 861611
+353 91 861612


Other materials required but not provided are:

- 6F or greater introducer sheath
- 0.018" (0.45mm) guidewire
- Sterile heparinized saline (for system flushing)

PREPARATIONS FOR USE

Review the schematic drawing of the PRONTO-Short catheter below for terminology used in the following steps.

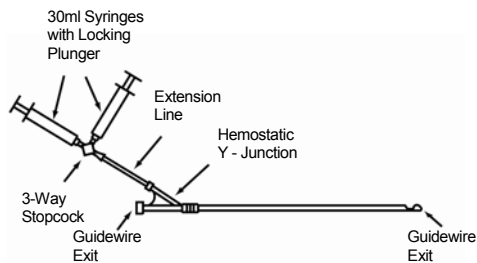1. Using sterile technique, open the pouch and transfer the products into the sterile field.
2. Remove the catheter from the carrier tube and inspect for any bends or kinks.
3. Thoroughly back flush the wire lumen with heparinized saline.
4. Draw 5ml of heparinized saline into both 30ml syringes and connect to the 3-way stopcock. Attach the extension line to the catheter and flush the entire connection to remove all air from the catheter, extension, stopcock and syringes. Turn the stopcock so that it is "Off" to the catheter.
5. With the stopcock in the "Off" position to the catheter, fully retract the plunger on one 30ml syringe. Twist the plunger to lock the syringe in the vacuum position. Repeat with the second syringe.

Precaution: Check that all fittings are secure so that air is not introduced into the extension line or syringes during extraction.

DEPLOYMENT PROCEDURE

Deploy the PRONTO-Short according to the following steps:

1. Cannulate the vessel using standard technique and a properly prepared 6F or greater introducer sheath.
2. Insert the guidewire into the desired vascular space.
3. Back-load the PRONTO-Short catheter onto the guidewire and advance it over the guidewire until the wire exits the proximal end of the Y-junction.
4. Secure the proximal segment of the guidewire and insert the PRONTO-Short catheter through the introducer sheath and into the desired vascular space. Stop advancement of the catheter if any resistance is encountered.
WARNING: Never advance or withdraw an intravascular device against resistance until the cause of the resistance is determined by fluoroscopy. Movement of the catheter or guidewire against resistance may result in separation of the catheter or guidewire tip, damage to the catheter, or vessel perforation.
5. After fluoroscopically confirming catheter position, open the stopcock to begin extracting into the first syringe. Slowly advance the catheter distally from the introducer sheath. Blood will enter the syringe until all of the vacuum is gone. Should aspiration not begin filling the syringe within 5 seconds, remove the catheter while still extracting. Outside the

patient, either flush the extraction lumen or use a new catheter.

WARNING: If flow into the syringe stops or is restricted, do NOT attempt to flush the extraction lumen while the catheter is still inside the patient vasculature. Intravascular thrombus delivery, a thromboembolic event and/or serious injury or death may result. Remove the catheter while the suction is on, and once outside the patient, either flush the extraction lumen or use a new catheter.

6. When the first syringe becomes full, open the stopcock to the second syringe to continue extraction, if desired.
NOTE: If air is noted in either syringe during extraction, a leak may be present in the system. Remove from the body and re-flush. Turn the stopcock "Off" to the catheter, tighten all luer connections, remove all air from the syringes, and repeat the extraction. If air is still noted, remove the catheter, obtain a new catheter and repeat the procedure.
7. After completing the extraction process, turn the stopcock to the "Off" position and remove the catheter.
8. Blood and thrombus extracted into the syringe may be filtered for subsequent laboratory analysis using the 70 micron filter basket. To use the filter basket, wet the mesh to allow fluid flow, and filter the extracted blood through the filter basket.

PACKAGING & STORAGE

The PRONTO-Short catheter has been sterilized with ethylene oxide.

STERILE EO

Handle with care.

LIMITED WARRANTY

Vascular Solutions, Inc. warrants that the PRONTO-Short extraction catheter is free from defects in workmanship and materials prior to the stated expiration date. Liability under this warranty is limited to refund or replacement of any product which has been found by Vascular Solutions, Inc. to be defective in workmanship or materials. Vascular Solutions, Inc. shall not be liable for any incidental, special, or consequential damages arising from the use of the PRONTO-Short extraction catheter.

Damage to the product through misuse, alteration, improper storage, or improper handling shall void this limited warranty.

No employee, agent, or distributor of Vascular Solutions, Inc. has any authority to alter or amend this limited warranty in any respect. Any purported alteration or amendment shall not be enforceable against Vascular Solutions, Inc.

THIS WARRANTY IS EXPRESSLY IN LIEU OF ALL OTHER WARRANTIES, EXPRESSED OR IMPLIED, INCLUDING ANY WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE OR ANY OTHER OBLIGATION OF VASCULAR SOLUTIONS, INC.

PATENTS AND TRADEMARKS

International and U.S. patents pending.

PRONTO® is a trademark of Vascular Solutions, Inc.

See the International Symbols Glossary on page 23.

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©2011 Vascular Solutions, Inc.
Pronto® Vascular Solutions, Inc.'in bir ticari markasıdır.