GOPHER™

SUPPORT CATHETER

English/Instructions for Use	7
Česky/Návod k Použití.....	9
Dansk/Brugsanvisning	11
Nederlands/Gebruiksaanwijzing	13
Suomi/Käyttöohje.....	15
Français/Mode D'emploi	17
Deutsch/Bedienungsanleitung	20
Ελληνικά/Οδηγίες Χρήσης	22
Italiano/Istruzioni per l'uso.....	25
Norsk/Bruksanvisning	27
Język polski/Instrukcja obsługi.....	29
Português/Instruções para utilização.....	32
Español/Instrucciones de Uso	34
Svenska/Bruksanvisning.....	36
Türkçe/Kullanma Talimatı	38

Manufactured by:

Vascular Solutions, Inc.
6464 Sycamore Court
Minneapolis, MN 55369 USA
Tel: +1 763-656-4300
Fax: +1 763-656-4250
www.vascularsolutions.com

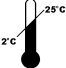
Authorized representative
of the European Community:

TheraGenesis GmbH
Bruehlstrasse 50
76297 Stutensee, Germany
Tel: +49 (0) 72 4995 2063
Fax: +49 (0) 72 4995 2064

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Instructions For Use

USA CAUTION

Federal (USA) law restricts this device to sale by or on the order of a physician.

CAUTION

The GOPHER Support Catheters should be used by physicians with adequate training in the use of the device. The device is provided sterile for single use only. Reuse of single-use device creates a potential risk of patient or user infections. Contamination of the device may lead to illness or serious patient injury.

DEVICE DESCRIPTION

The GOPHER Support Catheters are single lumen catheters designed for use in the arterial vasculature. The catheters provide support for guidewires during interventional procedures and allow for the exchange of one distally located guidewire for another while maintaining access to distal vasculature. The 3F GOPHER catheter is designed to assist in the passage of the catheter through lesions by rotating the catheter in a clockwise direction. The 3F model (5610) has a threaded, stainless steel distal tip.

The GOPHER catheters each have a radiopaque distal tip and two positioning marks located at 95cm (single mark) and 105cm (double marks) from the distal tip, respectively. The proximal end of the catheters each incorporate a torque device, strain relief and a luer-lock guidewire entry port for flushing.

The GOPHER catheters are designed to be used with guiding catheters and guidewires with the following dimensions:

Maximum Guidewire Diameter	0.014" (0.36mm)
Minimum Guide Catheter I.D.	0.058" (1.47mm)
Gopher Catheter Product Features	
Guidewire Lumen I.D.	0.017" (0.43mm)
Proximal Catheter O.D.	3F (0.99mm)
Distal Catheter O.D.	2F (0.66mm)
Distal Catheter Tip O.D.	
3F model 5610	3F (0.99mm)
Average Hand Injection Flow Rates Achieved (Results may vary.)	
Saline	0.590ml/sec
76% Ionic Contrast	0.051ml/sec

INDICATIONS

The GOPHER catheters are intended to be used in conjunction with steerable guidewires in order to access discreet regions of the arterial and/or coronary vasculature. The catheters may be used to facilitate placement and exchange of guidewires and other interventional devices, and to subselectively infuse/deliver diagnostic or therapeutic agents.

CONTRAINDICATIONS

The GOPHER catheters are contraindicated for high-pressure injections and for use in the cerebral vasculature.

WARNINGS

Do not reuse, reshape or re-sterilize the device. Re-sterilization or reshaping may change the physical characteristics of the material and should not be attempted.

The GOPHER catheters have not been tested for pressure injections. If a guidewire cannot be passed through the catheter, do not attempt to resolve the blockage by flushing the catheter *in vivo*. Catheter rupture and arterial injury could result. Identify and resolve the cause of the blockage, or replace the catheter with a new one.

Never advance, withdraw or rotate an intravascular device against resistance until the cause of the resistance is determined by fluoroscopy. Movement of the catheter or guidewire against

resistance may result in separation of the catheter or guidewire tip, damage to the catheter or vessel perforation.

Do not advance the GOPHER catheter ahead of the guidewire, as vessel perforation may result.

Never exceed five proximal rotations of the GOPHER catheter in the same direction if the distal tip is not rotating within a lesion or stenosis under fluoroscopy. Excess rotation may result in separation of the catheter, damage to the catheter or vessel perforation.

Do not use the catheter hub to torque the device. Always rotate the GOPHER catheter by using the torque device that is provided.

Do not use the GOPHER catheter in the cerebral vasculature.

COMPLICATIONS

As with all catheterization procedures, complications may occur when using the GOPHER catheter. These may include:

- local or systemic infection
- intimal disruption
- arterial dissection
- perforation and vessel rupture
- arterial thrombosis
- distal embolization of blood clots and plaque
- myocardial infarction
- arterial spasm
- catheter fracture with tip separation and distal embolization

PRECAUTIONS

The GOPHER catheter deployment procedure should be performed by physicians thoroughly trained in percutaneous, intravascular techniques and procedures.

Do not use the catheter if the packaging has been damaged.

Inspect the catheter prior to use for any bends or kinks. Do not use a damaged catheter because vessel damage and/or inability to advance or withdraw the catheter may occur.

Care should be taken not to crush the catheter. Excessive tightening of a hemostatic valve onto the catheter shaft may result in damage to the wire lumen and difficulty while exchanging wires.

The entire catheter lumen should be thoroughly flushed with heparinized saline prior to use to prevent clot formation.

Precautions to prevent or reduce clotting should be taken when any catheter is used in the vascular system. Use of systemic heparinization and heparinized sterile solution should be considered.

Exercise care while handling of the catheter during a procedure to reduce the possibility of accidental breakage, bending or kinking.

When the catheter is in the body, it should be manipulated only under fluoroscopy. Do not attempt to move the catheter without observing the resultant tip response.

Use caution in situations in which manipulation of the GOPHER catheter results in corresponding guidewire movement, as vessel or guidewire damage may result.

CLINICAL PROCEDURE

The following instructions provide technical direction but do not obviate the necessity of formal training in the use of the GOPHER catheters. The techniques and procedures described do not represent ALL medically acceptable protocols, nor are they intended as a substitute for the physician's experience and judgment in treating any specific patient.

Each GOPHER support catheter includes the following components:

- Single-use disposable catheter
- Dispenser coil with flushing luer

Other materials required but not provided are:

- Guiding catheter with an I.D. appropriate for the model of GOPHER catheter being used and fitted with a rotating hemostatic valve (RHV) (Tuohy-Borst type)
- Guidewire with a diameter appropriate for the model of

GOPHER catheter being used

- 10ml syringe (for flushing the dispenser coil and the catheter lumen)
- Sterile heparinized saline (for system flushing)

PREPARATIONS FOR USE

1. Carefully inspect the GOPHER catheter packaging and components for damage prior to use. Utilizing sterile technique, remove the GOPHER catheter dispenser coil from its packaging and transfer it to the sterile field.
2. Attach a 10ml syringe filled with sterile, heparinized saline to the luer-lock guidewire entry port of the GOPHER catheter and thoroughly flush the catheter.
3. Attach a 10ml syringe to the flushing luer and completely flush the coil.
4. Remove the GOPHER catheter from the dispenser coil and inspect it for any bends or kinks.

DEPLOYMENT PROCEDURE

The following GOPHER catheter deployment steps assume a standard PTCA protocol using the following items: a guiding catheter, an inserted exchange length guidewire and a rotating hemostatic valve (RHV) (Touhy-Borst type).

As with any interventional procedure, proper anticoagulation and anti-platelet therapy should be administered prior to beginning.

When not in use during this procedure, wipe the GOPHER catheter with a sterile gauze pad saturated with heparinized saline, flush the guidewire delivery lumen well and store in the dispenser tube in a saline bath.

Note: Familiarity with traditional guidewire exchange techniques is required for successful use of the GOPHER catheter in performing a guidewire exchange.

GOPHER DEPLOYMENT STEPS

1. Backload the distal tip of the GOPHER catheter onto the proximal end of the exchange length guidewire that is already in place in the distal vasculature.
2. Carefully advance the catheter until the marker band at the distal tip of the GOPHER catheter is visible in the desired distal vascular space.

WARNING: Never advance, withdraw or rotate an intravascular device against resistance until the cause of the resistance is determined by fluoroscopy. Movement of the catheter or guidewire against resistance may result in separation of the catheter or guidewire tip, damage to the catheter or vessel perforation.

3. If resistance is encountered while advancing the GOPHER catheter, rotate the catheter in a clockwise direction while continuing to advance.

WARNING: Never exceed five proximal rotations of the GOPHER catheter in the same direction if the distal tip is not rotating within a lesion or stenosis under fluoroscopy. Excess rotation may result in separation of the catheter, damage to the catheter or vessel perforation.

NOTE: The text on the printed strain relief may be used as an indicator of the number of proximal rotations of the catheter shaft.

4. If guidewire exchange is desired, after fluoroscopically confirming that the GOPHER catheter tip is positioned in the desired vascular space, slowly remove the guidewire.
5. Insert the new guidewire into the luer-lock guidewire entry port. Advance the guidewire until it exits the catheter tip.
6. Under fluoroscopy, ensure that the guidewire is sufficiently advanced into the distal vasculature. Fix the guidewire using standard guidewire exchange techniques and carefully withdraw the catheter. If resistance is encountered while removing the GOPHER catheter, rotate the catheter in a counter-clockwise direction while withdrawing.

PACKAGING & STORAGE

The GOPHER catheter has been sterilized with ethylene oxide.

STERILE EO

Store in a cool, dry place. Keep away from sunlight.

LIMITED WARRANTY

Vascular Solutions, Inc. warrants that the Gopher support catheter is free from defects in workmanship and materials prior to the stated expiration date. Liability under this warranty is limited to refund or replacement of any product that has been found by Vascular Solutions, Inc. to be defective in workmanship or materials. Vascular Solutions, Inc. shall not be liable for any incidental, special or consequential damages arising from the use of the Gopher support catheter. Damage to the product through misuse, alteration, improper storage or improper handling shall void this limited warranty.

No employee, agent or distributor of Vascular Solutions, Inc. has any authority to alter or amend this limited warranty in any respect. Any purported alteration or amendment shall not be enforceable against Vascular Solutions, Inc.

THIS WARRANTY IS EXPRESSLY IN LIEU OF ALL OTHER WARRANTIES, EXPRESSED OR IMPLIED, INCLUDING ANY WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE OR ANY OTHER OBLIGATION OF VASCULAR SOLUTIONS, INC.

PATENTS AND TRADEMARKS

International and U.S. patents pending.

Gopher™ is a trademark of Vascular Solutions, Inc.

See the International Symbols Glossary on page 2.

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Gopher™ Vascular Solutions, Inc.'in ticari bir markasıdır.