## INFLATION DEVICE FLAMINGO

**Sterile (sterilized with ethylene oxide). Do not re-sterilize. For single use. Do not use if the packaging is opened or damaged. Use before the expiry date shown on the packaging. Use with balloon dilatation catheters. Read the instructions before use. Store away from light in a cool and dry place.**

**Caution:** Federal (U.S.A) law restricts this device to use or sale by or on the order of a physician.

**Description:** Flamingo is an inflation device for single use. It consists of a 20 cc syringe body with a plunger which may be engaged by a control button, a rotatory handle, a manometer and a high pressure connector with a rotating male luer lock. The manometer is graduated between vacuum and 30 atm with an equivalent scale in PSI. There are two gripping choices by changing the manometer's position.

**Accessories may include:**

- Alligator torque device,
- Stopcock (435 psi).

**Indications:** Flamingo Inflation Device is intended for use during cardiovascular procedures to create, maintain and monitor pressure in the balloon catheter.

**Operation:** The plunger is free to move when the control button is rocked towards the back of the device (opposite point of manometer). To lock the plunger, push completely on the control button towards the manometer. Turn the handle clockwise to deliver the mixture of contrast media. The estimated pressure is indicated on the manometer.

**Preparation:**

1. Prepare the mixture of contrast media for balloon catheter following the manufacturer's recommendations for the contrast media.
2. Check the plunger is unlocked.
3. Hold the Flamingo with its manometer downwards and insert the end of connector into the contrast solution to fill the syringe.
4. Hold the device with the manometer upwards to purge the air contained in the syringe and in the connector.
5. Connect the Flamingo to the balloon catheter.
6. To inflate the balloon catheter, point the manometer downwards and turn **slowly** the handle clockwise up to posting of desired pressure. This pressure is kept due to the locking system.
7. To deflate the balloon catheter, unlock the plunger and pull completely the handle.

**Precautions for use:** The Flamingo must be used by doctors. Before each inflation, confirm the needle of the manometer is set to the zero position +/- 1 atm. When releasing the deflation, it is recommended to go with the handle. If the the control knob can't return in unlocked position, turn the handle counterclockwise to deflate the balloon catheter. Refer to the balloon catheter instructions for use (precautions for use, maximum pressure, etc).

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