




# DUETT™ PRO

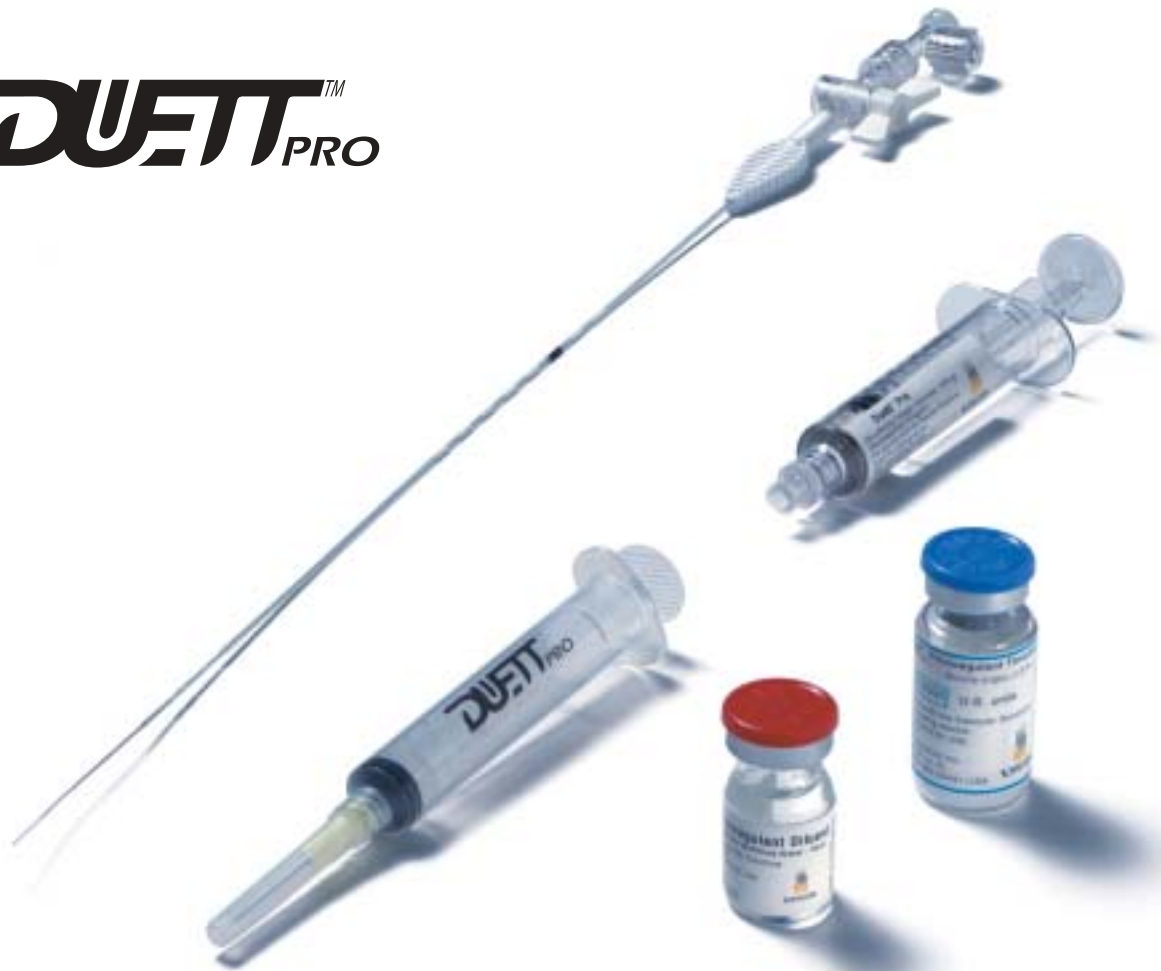

Model Number	Description	Case Quantity
1010	Duett™ Pro	10*

\* Devices are packaged in quantities of 10 units per box. Individual units are not sold separately.



2495 Xenium Lane North  
Minneapolis, MN 55441 USA  
Customer Service: 1.888.240.6001  
Office: 763.656.4300 Fax: 763.656.4251  
[www.vascularsolutions.com](http://www.vascularsolutions.com)


CAUTION: Federal law (U.S.A.) restricts this device to sale by or on the order of a physician. Please see the Duett Pro and Diagnostic Duett Pro sealing device Instructions for Use for a complete listing of the indications, contraindications, warnings and precautions.

Vascular Solutions Duett™ is a trademark of Vascular Solutions, Inc.



Introducing  
**DUETT<sup>TM</sup>**  
**PRO**

Simple, precise, efficient.

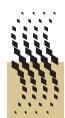

**vascular**  
SOLUTIONS







# Simple, precise, efficient.

**What makes it a "Pro"?** Duett Pro represents the next generation of Duett technology. Using the same proven Duett flowable procoagulant, the Pro line enhances the Duett catheter and simplifies the deployment steps. The Duett Pro's new catheter design features an efficient visual inflation indicator and precise sleeve markers for catheter location. Simple, Precise and Efficient – the Duett Pro delivers high-performance vascular sealing with consistency and ease.



## Simplified Self-Wrapping Balloon

Following procoagulant delivery, catheter withdrawal is as simple as 'deflate and remove'. During deflation, the Duett Pro balloon automatically twists and rewraps to an extremely low profile, allowing immediate removal without covering or extending. Removal of the Duett Pro catheter is now the same as removing any coronary or peripheral catheter.

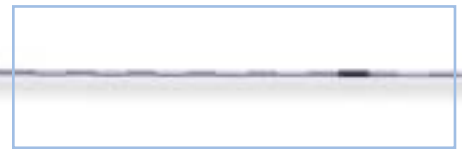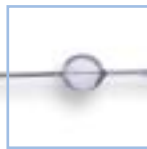

## Precise Sleeve Markers

For precise positioning of the catheter through the introducer sheath, white sleeve markers 5 mm in length are positioned 5 mm apart on the Duett Pro sleeve.





## Efficient Balloon Inflation Indicator

When inflating the Duett Pro catheter, a new inflation relief valve provides an efficient visual confirmation of proper inflation pressure. As the sealing balloon reaches its proper diameter at 2 atm of pressure, fluid begins to flow from the inflation relief valve, providing a clear visual indicator with no risk of over-pressurizing the system. The Duett Pro provides efficient, consistent balloon inflation.



## Ergonomic Sleeve Hub

Designed for ease of use, the Duett Pro sleeve hub is lightweight and ergonomic. A 'quick connect' feature provides a secure connection of the sleeve hub to the base during catheter deployment.